

# **Commode Chairs**

Shower commode chairs

# Discover the benefits



# **Commode Chair**

The Commode Chair is a flexible attendant propelled shower commode chair that gives carers the best possible access for personal care tasks. It is easy to manoeuvre and can be customised to many different user needs.



# **Unique** seat

The seat has a rear opening, giving the carer easy access when assisting the user with personal care tasks. Additionally, the "Easy Clean" friction pattern on the seat surface is unique for Commode and provides the perfect balance between friction and cleanliness.

### Comfortable relief

For sensitive users, extra-soft seat cushions are available, providing both support and comfortable relief. The soft and quick-drying backrest and the curved footrests of the Commode, increase user comfort further.

# **Highly Adaptable**

The Commode has a generous maximum user weight of 160 kg as standard and is height adjustable between 45-60 cm. The armrests are height and width adjustable, 6 cm in each direction. The swing away footrests are height adjustable and are also offered in a shorter version, e.g. when higher soft seats are used.

### Commode XL

A configuration with a wider back frame is available for larger users, providing an extra 6 cm between the armrests. The XL-back frame is also available as an accessory.

Standard: backrest, seat, armrests, footrests and 4 lockable castors

#### Weight

16,6 kg

#### **Dimensions**

Total width: 57/58/64 cm Width between arm supports: 42/48/54 cm Width between arm supports with XL back: 48/54/60 cm



#### **Materials**

Chassis: powder painted steel Seat/Armrests: polypropylene Backrest: polyester/Velcro Footrest: polypropylene Castors: polyamid/TPE, lockable, ball bearing









Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.

The product can be safely cleaned in a cabinet washer-disinfector at  $85^{\circ}\text{C}$  for 3 minutes.

Max user weight: 160 kg / 353 lbs

# Commode Mobil 24"-2

Commode Mobil 24"-2 is a self-propelled model, designed for the independent user.



# Easy to manoeuvre

The shower commode chair is simple to use and easy to manoeuvre. The rear wheels are positioned for maximum control and are easy to reach. Flip up armrests and generous space in front of the wheels, allow for easy side transfers. The relatively even shape of the seating area also simplifies side transfers.

#### Comfortable seat

To facilitate access, the seat has a rear opening and a hygiene recess in front. The seat is designed to provide comfort, regardless of the users posture or pelvis width. For additional comfort, ergonomic soft seats are available.

# Fits many user needs

Commode Mobil 24"-2 is adaptable to fit many user needs and sizes. It has a generous maximum user weight of 160 kg as standard and adjustable seat-to-floor height (45-60 cm). The armrests are adjustable both in height and width, and the soft backrest can be adjusted for positioning and comfort.

For larger users, an extra wide back frame is offered, providing an additional 6 cm between the armrests.

Standard: backrest, seat, armrests, footrests and 4 lockable castors

#### Weight

25,1 kg

#### **Dimensions**

Total width: 72 cm
Width between arm supports: 42/48/54 cm



#### **Materials**

Chassis: powder painted steel Seat/Armrests: polypropylene Backrests: polyester/Velcro Footrests: polypropylene Castors: polyamid/TPE, lockable, ball bearing Rear wheels: solid tyres, polyamid, ball bearing Brakes: polyamid, stainless steel









Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.

The product can be safely cleaned in a cabinet washer-disinfector at  $85\,^{\circ}\text{C}$  for 3 minutes.

Max user weight: 160 kg / 353 lbs



# Commode Mobil Tilt-2

The Commode Mobil Tilt-2 is designed with both carer and user needs in mind. It handles the multitude of challenges faced by caregivers in their daily work and provides the comfort and safety users need to feel secure.



# Unique tilt-in-space function

The Commode Mobil Tilt-2, has a unique tilt feature that gives increased security for the user and an ergonomic working position for the carer. It's +5° to -35° tilt function can be operated from either side of the chair. This allows the carer to reassure the user when tilting.

### Ease of access and comfort

With ease of access around the seat area, personal care routines are made easier for the carer and can be performed with dignity for the user. For sensitive users, extra-soft, seat cushions are available. The soft and quick-drying backrest and curved footrests of the Commode Mobil-2, add to user comfort.

# Fits many users

The shower commode chair is adaptable to fit many user needs and sizes. It has a generous maximum user weight of 160 kg as standard, and seat-to-floor height adjustability (45-60 cm). The armrests are adjustable in height and width, and the height adjustable, swing away footrests are also offered in a shorter version.

Standard: headrest, backrest, seat, armrests, footrests and 4 lockable castors

#### Weight

20,4 kg

#### **Dimensions**

Total width: 57/56/64 cm Width between arm supports: 42/48/54 cm



#### Materials

Chassis: powder painted steel Seat/Armrest: polypropylene Backrest: polyester/Velcro Footrest: polypropylene

Castors: polyamid/TPE, lockable, ball bearing









Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.

The product can be safely cleaned in a cabinet washer-disinfector at 85°C for 3

Max user weight: 160 kg / 353 lbs

# Features - for all models



#### Seat with rear opening

Providing ease of access for the carer and comfort for the user. The unique surface is especially developed for Commode and has the perfect balance between friction and cleanli-ness, non-slip and hygienic!



#### Unique and safe tilting function

Can be operated from either side of the chair enabling a good working position for the carer and reassurance for the user during tilting. (Mobil Tilt-2 only)



#### Adjustable armrests as standard

Easily adapted to various user needs. Adjustable both in height and in width, +/- 6 cm in each direction.



#### Multi-adjustable headrest

The headrest is designed to provide maximum comfort and flexibility and can be adjusted to suit a range of user needs. (Mobil Tilt-2 only)



#### Easy to manoeuvre

The position of the rear wheels makes it easy to propel the chair forward, as well as simplifying side transfers. (Mobil 24"-2 only)



#### Adjustable backrest

Easily adjusted for posture, body shape and maximum comfort. Fitted with two fasteners around the pushbars. Soft and quick-drying material.



#### Adjustable seat height

Commode Mobil 24"-2 can be easily set at various fixed heights with tool free adjustment ensuring a safe and comfortable height.



#### **Ergonomic footrest**

Curved footplates provide instep and arch support, ensuring comfort, stability and relaxation.



# Accessories - for all models

(See washing instructions on page 11)



# Soft comfort seat, 4 cm rear opening

80209519

Ergonomically shaped with substantial softness providing comfortable relief and support. Designed with a recess at the front. Material: Polyurethane



# Soft comfort seat, 2 cm rear opening

80209518

Designed with substantial softness and round shapes for maximum comfort. Designed with a recess at the front.





#### Soft comfort seat, 4 cm

Small opening 15 cm (6"): 80209507 Large opening 18 cm (7"): 80209506 Ergonomically shaped soft comfort seats with recesses at the front. Height is 4 cm at the lowest point.

Material: Polyurethane.



#### Soft comfort seat, 2 cm

Opening 18 cm: 80209306 Shaped, soft comfort seat with recess at the front. Height is 2 cm at the lowest point.

Material: Polyurethane.



#### Seat cover

80209434

A comfortable, closed seat when you want to cover the hygiene opening. Can be used in the shower. Height 2 cm.

Material: Polyurethane.



#### Soft cushion with standard opening

80209065

Foam cushion with a cover made of a water-repellent fabric. Height 2 cm. Material: Polyether/coated plastic.



#### Soft cushion with oval opening

80209067

Foam cushion with a cover made of a waterrepellent fabric. Height 4 cm. Material: Polyether/coated plastic.



#### **ROHO®** Protect cushion

80209245

Single chamber inflatable cushion with excellent pressure relieving features. Intended for users who are extra sensitive to pressure. Recess width: 21 cm.

Material: Neoprene



#### Headrest

80209447

Multi-adjustable headrest with shaped pillow. Adjustable in height, depth, sideways and in angle. Complete with bracket.

Material: Pillow: Polyurethane Brace: Stainless steel



#### **Comfort cover**

80209455

Soft back with integrated armrest cushions. The comfort padding provides a soft support while the gripping surface on the armrest is kept free.

Material: Polyethylene.



#### Side support

80209509

Soft back with integrated armrest cushions. Material: Polyester fabric, Velcro,

Polyethylene.



#### XL-back

80209454

For larger users, offering more room for back and shoulders. Increases the width between the armrests by 6 cm to a total width of 48/54/60. (XL-back does not fit Commode Mobil Tilt-2)
Material: Polyester, Powder coated steel, stainless steel. polypropylene

# Accessories – for all models

(See washing instructions on page 11)



#### Soft back pad

80209448

Provides the user with warmth and softness. Used inside the armrests, which makes it possible to fold up the armrests and create accessibility for side transfer.

Material: Polyethylene.



#### Armrest pad

80209508

Used to create softness around the armrests. Provides a warm, soft surface to support the arms while keeping the gripping surface free, making it safe to stand up. Sold in pairs. Material: Polyethylene.



#### **Armrest Hemiplegia**

80209035

Provides support for the forearm with its contoured shape. Equipped with a heatreflecting material on the outside. Fits the left and right armrests.

Material: Polyethylene



#### **Armrest lock**

80209445

A locking pin to be used when you want to lock the armrest in the folded-down or foldedup position.

Material: Stainless steel.



#### Splash guard

80209430

A soft protection against urine splashes at the front. Attaches to the front recess of the seat and can easily be removed.

Material: Polyurethane



#### Cross har

60.5 cm 80303034 66.5 cm 80209452

A safe support to hold on to or it can be used as a strain support. Attaches to the armrests. Material: Aluminium, polyethylene.



#### Hip/chestbelt upholstered

80209524

2-part belt for positioning that can be used either as hip- or chest belt. Adjustable with 3 buckles with up to 65-130 cm length. The material is comfortable and quick-drying and provides firm support when positioning.



#### Hip belt

80209440

Easy to adjust with a plastic buckle. Attaches around the back tubes in the rear with velcro

Material: Polyester



#### Chest belt

80209442

Easy to adjust and attaches around the straps of the back support upholstery. Use with a hip belt is recommended.

Material: Polyester



#### Pan with lid

Pan with lid and handle: 80209255 Unique interior shape that breaks the stream

and prevents splashes. The lid has a handle that locks onto the pan and ensures that the content does not spill.

Material: Polypropylene



#### Pan

Pan without lid 80209254

Lid with handle 80209256

Unique interior shape that breaks the stream

and prevents splashes.

Material: Polypropylene



#### Pan holder

80209257

Suitable for both the Etac pan and traditional hospital pans. Used together with the attachment holders, which are ordered separately.

Material: Polypropylene



#### Attachment holder

80209435

To be mounted firmly under the seat enabling the use of pan holders. Fitted with a

screwdriver. Complete set. Material: Polyamide



#### Low mounting of pan/bucket

80209259

Used when you want the pan/bucket placed lower under the seat. Mounted with the pan or the bucket holders. Lowers the bed pan/ bucket holders by 6 cm.

Material: Polypropylene.

### Accessories – for all models

(See washing instructions on page 11)



#### **Bucket holder**

80209258

Used as a holder for pans/buckets or bed pans with higher edges. Used together with the attachment holders, which are ordered separately.

Material Polypropylene.



#### Bucket with handle and lid



#### Leg and calf support

80209437

80209443

Leg rest with an adjustable angle that can be used for both left and right legs. Comes with 4 fixed-angle positions. Sold per piece. Material: Stainless steel and ABS-plastic.

Footrest for leg and calf support

Kit with separate foot plate to be mounted

together with leg and calf support.





#### **Anti-tip Commode Mobil Tilt-2**

Anti-tip for Commode Mobil Tilt-2: 80209459

Provides protection against the chair tipping backwards. Increases safety when the centre of gravity falls far to the back.

Material: Polypropylene and Aluminium



#### **Amputee legrest**

Material: ABS plastic

80209510

Adjustable in height, length and angle. Can be used on both left and right sides. If the user is an amputee we recommend using anti-tippers.

Material: Stainless steel and polyurethane



#### **Anti-tip Commode Mobil-2**

Anti-tip Including attachment: 80209439 Anti-tip excluding attachment: 80209438 Provides protection against the chair tipping backwards.

Material: Polypropylene and Aluminium



#### Heel straps

80209432

Mounted on foot plates. Prevents the feet from sliding backwards. Adjustable in length. Sold as a pair.

Material: Polyester



#### Tilter

80209449

Facilitates transfer over e.g. thresholds or ramps. Sold complete with bracket. Material: Polypropylene and Aluminium



#### Rear wheel kit 24"

80209436

24" rear wheels with solid tyres and built-in tipping guards. Easy to mount. Includes attachment and brake.

Material: Polyamide, stainless steel, polyurethane aluminium



#### **Short footrests**

80209520

A short footrest that swings sideways. Can be adjusted 12.5 cm and provides a footrest length of 24 - 36.5 cm. The foot plate has a rounded shape. Sold as a pair.

Material: stainless steel and polypropylene



#### Footrests with foldable footplates

Adjustable for those who need a more open foot angle. Can be equipped with a heel band. Sold as a pair.

Material: Stainless steel and polypropylene.

# **Washing instructions**

Soft Comfort seats and Seat cover, Headrest, Armrest lock, Splash guard, Pan, Pan holder, Attachment holder, Low mouning attachment, Bucket holder, Bucket, Anti-tip, Tilter, Footrests, Leg and calf support, Amputee legrest, Rear wheel 24" kit







Backrest upholstery, Hip/chestbelt upholstered, Hip belt, Chest belt, Heel strap



Soft cushions, Comfort cover, Soft back pad, Armrest pad, Armrest hemiplegia, Cross bar





ROHO® protect







### Washing symbols explained



Machine wash - max. temperature 60 °C / 140 °F



Clean the product with a solvent-free cleaning agent with a pH level between 6–9, or with a 70% disinfectant solution.



The product can be cleaned in a cabinet washer-disinfector at max. 85  $^{\circ}$ C / 185  $^{\circ}$ F for 3 minutes.



No dry cleaning



Wiping - the product can be wiped clean with a cleaning cloth



"Everyone should be able to live a free and independent life and pursue their dreams regardless of any physical circumstances"





